

CavityShield Varnish- Directions For Use:

- 1.** Teeth should be “toothbrush” clean. A prophylaxis is not required.
- 2.** Open package and use the brush to move any varnish that has adhered to the foil into the well.
- 3.** Use the bristle end of the brush to thoroughly mix the fluoride varnish.
- 4.** For best results, excessive saliva should be removed by blotting the tooth surfaces. The fluoride varnish will ‘set’ upon contact with water/saliva.
- 5.** Paint teeth evenly.
- 6.** The patient can leave immediately after the application of CavityShield.
- 7.** It is best for the patient to leave CavityShield in contact with tooth surfaces as long as possible, minimally for 4 hours. During this period, instruct the patient to avoid hard foods, brushing teeth and use of products containing alcohol (beverages, oral rinses). A thorough brushing and flossing will remove excess fluoride varnish from tooth surfaces following the treatment period.
- 8.** CavityShield can be removed from instruments with alcohol.